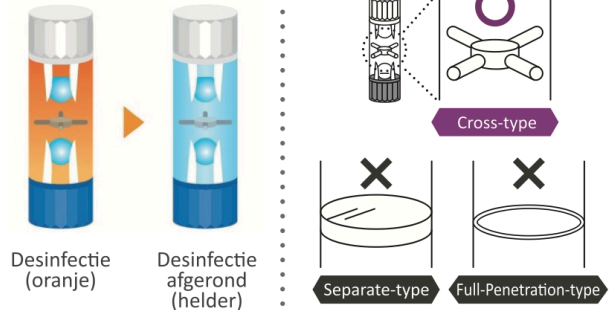
# cleadew GP

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Test method:  
Adhere denatured lysozyme deposits onto the lens and carry out care with cleadew GP. Dye the leftover deposits in red before confirming the leftover counts. (Ophtecs data)

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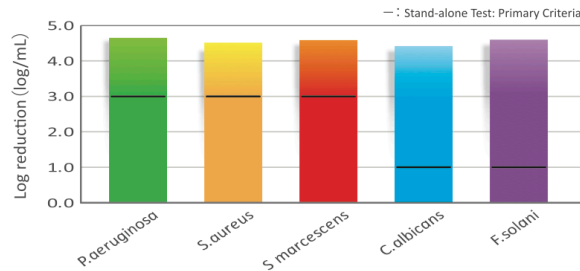
Test method:  
Adhere compounding lipid deposits onto the lens and carry out care with cleadew GP. The removal efficacy is measured and determined by the amount of leftover lipid deposits. (Ophtecs data)

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
Test method:  
According to stand-alone test (ISO14729),  $1.0 \times 10^5 \sim 10^6$  cfu/mL of the test strains are placed into cleadew GP and left aside for the required care time. The remaining live strains are counted afterwards. (Ophtecs data)

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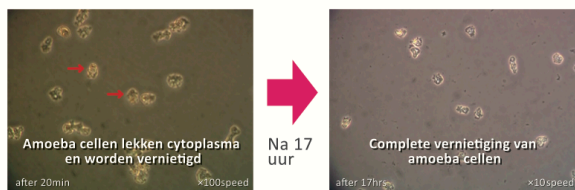
Test method:  
Acanthamoeba (trophozoite) is inoculated in disinfecting solution and neutralizing tablet of cleadew GP. After 4 hours, the solution containing acanthamoeba is introduced into E.coli-MY culture medium. After 17 hours, the condition of acanthamoeba is examined. (Ophtecs data)

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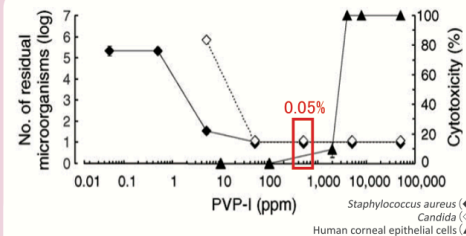Povidone-iodine has a high disinfecting efficacy against bacteria and viruses, while still maintaining a safety level of the corneal impact.


Povidone-iodine shows sufficient disinfecting efficacy between 50~500 ppm (left axis). However, a low cytotoxicity level is confirmed up to 2000 ppm (right axis).

cleadew GP contains 0.05% povidone-iodine. It maintains a high disinfecting efficacy and a low level of cytotoxicity.

R. Yanai et al. / Contact Lens & Anterior Eye 29 (2006) 85–91

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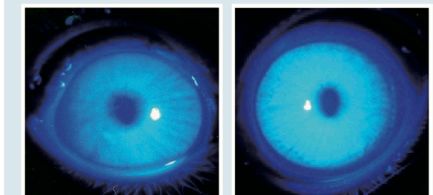Impact after addition of 0.05% povidone-iodine

Untreated eyes

After addition of 0.05% povidone-iodine

Test method:  
Add 0.05% of povidone-iodine into the eyes of the laboratory rabbit. After 5 minutes, apply fluorescent dye into the eyes and examine it with blue rays under a slit lamp. (Ophtecs data)