**IMPORTANT: Please read carefully and
keep this information for future use**

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**ACUVUE®
and
SUREVUE®
Brand Contact
Lenses**

**Package Insert/ Packungsbeilage /
Productbijlage / Prospektüs**

**CE
0086**

English

IMPORTANT: Please read carefully and keep this information for future use 3

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English

Important: Please read carefully and keep this information for future use. This Package Insert is intended for the Eye Care Professional, but should be made available to patients upon request. This insert contains important information about the correct usage of ACUVUE® and SUREVUE® Brand Contact Lenses, which are listed in Table 1 and contains information about adverse reactions and contraindications.

Johnson & Johnson Medical Ltd provides ACUVUE® and SUREVUE® Brand Contact Lenses Patient Instruction Guides that contain further information for contact lens wearers. The Eye Care Professional should provide the patient with the Patient Instruction Guide that pertains to the patient's prescribed lens.

Table 1

Lens type and Brand name	Intended Use & Wear Schedule			Inside-out indicator	Material	Polymer/water content (%)	Packaging Solution
	Daily Wear - Daily disposable	Daily wear Frequent replacement	Extended wear				
ACUVUE® Brand Spherical Contact Lenses - Visibility Tinted with UV Blocker							
ACUVUE®2® Brand Contact Lens		☉	☉		etafilcon A	42/58	①
1•DAY ACUVUE® Brand Contact Lenses	☉				etafilcon A	42/58	①
1•DAY ACUVUE® MOIST® Brand Contact Lenses	☉				etafilcon A	42/58	③
SUREVUE® Brand Contact Lenses		☉			etafilcon A	42/58	①
ACUVUE® ADVANCE® Brand Contact Lenses with HYDRACLEAR®		☉			galyfilcon A ④	53/47	②
ACUVUE® OASYS® Brand Contact Lenses with HYDRACLEAR® PLUS		☉	☉		senofilcon A ④	62/38	②
1•DAY ACUVUE® TruEye® Brand Contact Lenses	☉				narafilcon A ④	54/46	②
ACUVUE® ADVANCE® PLUS Brand Contact Lenses with HYDRACLEAR®		☉			galyfilcon A ④	53/47	②
1•DAY ACUVUE® DEFINE™ Brand Contact Lenses with LACREON®	☉				etafilcon A	42/58	③
ACUVUE® Brand Contact Lenses for ASTIGMATISM - Visibility Tinted with UV Blocker							
1•DAY ACUVUE® MOIST® Brand Contact Lenses for ASTIGMATISM	☉				etafilcon A	42/58	③
1•DAY ACUVUE® Brand Contact Lenses for ASTIGMATISM	☉				etafilcon A	42/58	①
ACUVUE® ADVANCE® Brand Contact Lenses for ASTIGMATISM with HYDRACLEAR®		☉			galyfilcon A ④	53/47	②
ACUVUE® OASYS® Brand Contact Lenses for ASTIGMATISM with HYDRACLEAR® PLUS		☉	☉		senofilcon A ④	62/38	②
ACUVUE® Brand Contact Lenses for PRESBYOPIA - Visibility Tinted with UV Blocker							
ACUVUE® BIFOCAL Brand Contact Lens		☉	☉		etafilcon A	42/58	①
ACUVUE® OASYS® Brand Contact Lenses for PRESBYOPIA with HYDRACLEAR® PLUS		☉	☉		senofilcon A ④	62/38	②

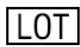
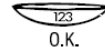
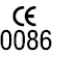
Key

Packaging Solution: ① Buffered saline ② Buffered saline with methyl ether cellulose ③ Buffered saline with povidone. **Material content:** ④ Lens material contains silicone and meets Class 1 UV absorbing standards with transmissibility of less than 1% UVB (280-315nm) and 10% UVA (316-380nm) radiation.

All etafilcon A products meet Class 2 UV absorbing standards with transmissibility of less than 5% UVB and 50% UVA radiation.

Symbols Key

The following symbols may appear on the label or carton of your ACUVUE® or SUREVUE® Brand Contact Lenses

SYMBOL		DEFINITION	SYMBOL	DEFINITION
		Consult Instruction Leaflet		Manufactured by or in
		Use By Date (expiration date)	See table 1 for list of Brands	
		Batch Code		Lens Orientation Correct
		Sterile Using Steam or Dry Heat		Lens Orientation Incorrect (inside Out)
DIA		Diameter	Lenses for ASTIGMATISM only	
BC		Base Curve	CYL	Cylinder Power
D		Dioptre (lens power)	AXIS	Axis
		UV-Blocking	ACUVUE® OASYS® Brand Contact Lenses for PRESBYOPIA with HYDRACLEAR® PLUS only	
		Quality System Certification Symbol	LOW	“Low” near ADD (+0.75 to +1.25 ADD)
		Peel Back Foil	MID	“Mid” near ADD (+1.50 to +1.75 ADD)
		Fee Paid for Waste Management	HGH	“High” near ADD (+2.00 to +2.50 ADD)
		CAUTION: US Federal law restricts this device to sale by or on the order of a licensed practitioner.	1•DAY ACUVUE® DEFINE™ Brand Contact Lenses with LACREON® only	
			ND	NATURAL SHINE™ for DARK EYES
			NL	NATURAL SHINE™ for LIGHT EYES
		“Identification mark” for paper containers and wrapping		“Identification mark” for composite materials

CONTENT

ACUVUE® and SUREVUE® Brand Contact Lenses are provided in sterile individual blister packages, with the lens immersed in buffered saline solution (see **Table 1** for Packaging Solution). **DO NOT USE** if the sterile blister package is opened or damaged.

INTENDED USE

ACUVUE® & SUREVUE® Spherical Brand Contact Lenses

Are intended for Daily Wear or Extended Wear, as shown in **Table 1**, for the optical correction of refractive ametropia (myopia and hyperopia) in phakic or aphakic persons with non-diseased eyes that may have 1.00D or less of astigmatism.

1•DAY ACUVUE® DEFINE™ Brand Contact Lenses with LACREON® are also intended to enhance or alter the appearance of the eye.

ACUVUE® Brand Contact Lenses for Astigmatism

Are intended for Daily Wear or Extended Wear, as shown in **Table 1**, for the optical correction of refractive ametropia (myopia and hyperopia) in phakic or aphakic persons with non-diseased eyes that may have 0.50D to 2.50D of astigmatism.

ACUVUE® Brand Contact Lenses for Presbyopia

Are intended for Daily Wear or Extended Wear, as shown in **Table 1**, for the optical correction of distance and near vision in presbyopic, phakic or aphakic persons with non-diseased eyes who have 0.75D or less of astigmatism.

Additionally, ACUVUE® OASYS® Brand Contact Lenses are also indicated for therapeutic use as a bandage lens for the following acute and chronic ocular conditions:

- For corneal protection in lid and corneal abnormalities such as entropion, trichiasis, tarsal scars and recurrent corneal erosion. In addition, they are indicated for protection where sutures or ocular structure malformation, degeneration or paralysis may result in the need to protect the cornea from exposure or repeated irritation.
- For corneal pain relief in conditions such as bullous keratopathy, epithelial erosion and abrasion, filamentary keratitis, and post-keratoplasty.

- For use as a barrier during the healing process of epithelial defects such as chronic epithelial defects, corneal ulcer, neurotrophic and neuroparalytic keratitis, and chemical burns.
- For post surgical conditions where bandage lens use is indicated such as post refractive surgery, lamellar grafts, corneal flaps, and additional ocular surgical conditions.
- For structural stability and protection in piggy back lens fitting where the cornea and associated surfaces are too irregular to allow for corneal rigid gas permeable (RGP) lenses to be fitted. In addition, the use of the lens can prevent irritation and abrasions in conditions where there are elevation differences in the host/graph junction or scar tissue.

ACUVUE® OASYS® Brand Contact Lenses when prescribed for therapeutic use may be worn for Daily Wear or Extended Wear.

All ACUVUE® and SUREVUE® Brand Contact Lenses have UV Blocking to help provide protection against transmission of harmful UV radiation to the cornea and into the eye.

WARNING: UV absorbing contact lenses are NOT substitutes for protective UV absorbing eyewear such as UV absorbing goggles or sunglasses because they do not completely cover the eye and surrounding area. You should continue to use UV absorbing eyewear as directed.

Note: Long-term exposure to UV radiation is one of the risk factors associated with cataracts. Exposure is based on a number of factors such as environmental conditions (altitude, geography, cloud cover) and personal factors (extent and nature of outdoor activities). All ACUVUE® and SUREVUE® Brand Contact Lenses have UV blocking to help provide protection against transmission of harmful UV radiation to the cornea and into the eye. However, clinical studies have not been done to demonstrate that wearing UV blocking contact lenses reduces the risk of developing cataracts or other eye disorders. Consult your Eye Care Professional for more information.

WEAR SCHEDULE

The wearing and replacement schedule should be determined by the Eye Care Professional. Patients may tend to over wear the lenses initially. The Eye Care Professional should emphasise the importance of adhering to the initial maximum wearing schedule. Regular checkups, as determined by the Eye Care Professional, are also extremely important.

Daily Wear - Daily Disposable

ACUVUE® Brand Contact Lenses prescribed for Daily Wear - Daily Disposable (less than 24 hours, while awake) as shown in **Table 1**, are intended to be worn once on a daily disposable basis and are to be discarded upon removal. **When used in this way, no cleaning or disinfection is required.**

1•DAY ACUVUE® TruEye® Brand Contact Lenses have not been developed for use with contact lens cleaners or disinfection systems. Lenses should be discarded after use. Start each wearing period with a fresh new lens.

Daily Wear - Frequent Replacement

ACUVUE® Brand Contact Lenses prescribed for Daily Wear - Frequent Replacement (less than 24 hours, while awake) as shown in **Table 1**, are to be discarded and replaced every two weeks.

SUREVUE® Brand Contact Lens for Daily Wear - Frequent Replacement (less than 24 hours, while awake) as shown in **Table 1**, are to be discarded and replaced every four weeks.

All ACUVUE® and SUREVUE® Brand Contact Lenses as shown in **Table 1**, under the heading of Daily Wear - Frequent Replacement are to be cleaned, rinsed and disinfected each time the lens is removed using a chemical disinfection system only.

Extended Wear

ACUVUE® Brand Contact Lenses prescribed for Extended Wear, (greater than 24 hours, including while asleep) as shown in **Table 1**, may be used continuously for up to 7 days/6 nights and should be discarded upon removal. When used in this way, **no cleaning or disinfection is required.**

It is recommended that the new contact lens wearer first be evaluated on a Daily Wear - Frequent Replacement schedule. If in the opinion the Eye Care Professional, the patient is determined to be an acceptable Extended Wear candidate, the Eye Care Professional is encouraged to determine a wear schedule based upon the response of the patient.

Once removed, it is recommended that the lens remains out of the eye for a period of rest overnight or longer. The Eye Care Professional should examine the patient during the early stages of Extended Wear.

CONTRAINDICATIONS

DO NOT USE ACUVUE® or SUREVUE® Brand Contact Lenses when any of the following conditions exist, when prescribing for refractive ametropia use.

- Acute or subacute inflammation or infection of the anterior chamber of the eye.
- Any eye disease, injury or abnormality that affects the cornea, conjunctiva or eyelids.
- Severe insufficiency of lacrimal secretion (dry eye).
- Corneal hypoesthesia (reduced corneal sensitivity).
- Any systemic disease that may affect the eye or be exaggerated by wearing contact lenses.
- Allergic reactions of ocular surfaces or adnexa that may be induced or exaggerated by wearing contact lenses or use of contact lens solutions.
- Allergy to any ingredient, such as mercury or Thimerosal in a solution which is to be used to care for the lenses prescribed on a frequent replacement wear schedule.
- Any active corneal infection (bacterial, fungal, protozoal or viral).
- If eyes become red or irritated.

For THERAPEUTIC USE, the Eye Care Professional may prescribe ACUVUE® OASYS® Brand Contact Lenses, to aid in the healing process of certain ocular conditions, which may include those cited above.

WARNINGS

(Daily Wear = less than 24 hours, while awake; Extended Wear = greater than 24 hours, including while asleep).

Proper use and care of contact lenses and lens care products, including lens cases, are essential for safe use. Problems from wearing contact lenses or using lens care products could result in serious injury to the eye.

Patients should be advised of the following warnings pertaining to contact lens wear†:

- Problems with contact lenses or lens care products could result in serious injury to the eye. Patients should be cautioned that proper use and care of contact lenses and lens care products, including lens cases, are essential for the safe use of these products
- Eye problems, including corneal ulcers, can develop rapidly and lead to loss of vision.
- Studies have shown that the risk of ulcerative keratitis is greater for extended wear contact lens users than for daily wear users.
- When daily wear users wear their lenses overnight (outside the approved indication), the risk of ulcerative keratitis is greater than among those who do not wear them overnight.

- The overall risk of ulcerative keratitis may be reduced by carefully following directions for lens care, including cleaning the lens case.
- Studies have shown that the risk of ulcerative keratitis among contact lens users who smoke is greater than among non-smokers.

If patients experience eye discomfort, excessive tearing, vision changes, redness of the eye or other problems are experienced, they should be instructed to immediately remove their lenses and the wearer should promptly contact their Eye Care Professional. It is recommended that contact lens wearers should see their Eye Care Professional routinely as directed.

† *New England Journal of Medicine*, September 21, 1989, 321 (12), pp. 773-783

PRECAUTIONS

What to do if problems occur

PLEASE ADVISE YOUR PATIENT TO REMOVE CONTACT LENSES IMMEDIATELY AND SEEK ADVICE FROM THEIR EYE CARE PROFESSIONAL.

Special Precautions for Eye Care Professionals

Due to the small number of patients enrolled in clinical investigation of lenses, all refractive powers, design configurations or lens parameters available in the lens material are not evaluated in significant numbers. Consequently, when selecting an appropriate lens design and parameters, the Eye Care Professional should consider all characteristics of the lens that can affect lens performance and ocular health, including oxygen transmissibility, wettability, central and peripheral thickness and optic zone diameter.

The potential impact of these factors on the patient's ocular health should be carefully weighed against the patient's need for refractive correction; therefore, the continuing ocular health of the patient and lens performance on the eye should be carefully monitored by the prescribing Eye Care Professional.

- Due to the reduction in light transmittance with cosmetically tinted lenses some patients may experience visual symptoms while wearing 1•DAY ACUVUE® DEFINE® Brand Contact Lenses with LACREON™. In addition, some patients may experience peripheral awareness due to the opaque iris pattern.
- Patients who wear the ACUVUE® and SUREVUE® Brand Contact Lenses to correct presbyopia using Monovision may not achieve the best corrected visual acuity for

either far or near vision. Visual requirements vary with the individual and should be considered when selecting the most appropriate type of lens for each patient.

- Fluorescein, a yellow dye, should not be used while the lenses are on the eyes. The lenses absorb this dye and become discolored. Whenever fluorescein is used in eyes, the eyes should be flushed with a sterile saline solution that is recommended for in-eye use.
- Eye Care Professionals should instruct the patient to remove the lenses immediately if the eyes become red or irritated.
- Eye Care Professionals should carefully instruct patients about the following care regimen and safety precautions.

Handling Precautions

- Before leaving the Eye Care Professional's office, the patient should be able to promptly remove lenses or should have someone else available who can remove the lenses for him or her.
- **DO NOT** use if the sterile blister package is opened or damaged.
- Always wash and rinse hands before handling lenses. Do not get cosmetics, lotions, soaps, creams, deodorants or sprays in the eyes or on the lenses. It is best to put on lenses before putting on makeup. Water-based cosmetics are less likely to damage lenses than oil-based products.
- Do not touch contact lenses with the fingers or hands if the hands are not free of foreign materials, as microscopic scratches of the lenses may occur, causing distorted vision and/or injury to the eye.
- Carefully follow the handling, insertion, removal, cleaning, disinfecting, storing and wearing instructions in the "Patient Instruction Guide" for ACUVUE® and SUREVUE® Brand Contact Lenses and those prescribed by the Eye Care Professional.
- Always handle lenses carefully and avoid dropping them.
- Never use tweezers or other tools to remove lenses from the lens container unless specifically indicated for that use. Pour the lens and the packing solution into the hand.
- Do not touch the lens with fingernails.
- Close supervision is necessary for the Therapeutic use of all ACUVUE® OASYS® Brand Contact Lenses. Ocular medications used during treatment with a bandage lens should be closely monitored by the Eye Care Professional. In certain ocular conditions, only the Eye Care Professional will insert and remove the lenses. In these cases, patients should be instructed not to handle the lenses themselves.

Lens Wearing Precautions

- If the lens sticks (stops moving) on the eye, follow the recommended directions in "Care for a Sticking Lens". The lens should move freely on the eye for the continued

health of the eye. If non-movement of the lens continues, the patient should be instructed to immediately consult his or her Eye Care Professional.

- Never wear lenses beyond the period recommended by the Eye Care Professional.
- If aerosol products, such as hair spray, are used while wearing lenses, exercise caution and keep eyes closed until the spray has settled.
- Avoid all harmful or irritating vapours and fumes while wearing lenses.
- The Eye Care Professional should warn the patient about wearing lenses during sporting activities, especially swimming and other water sports. Exposing contact lenses to water during swimming or while in a hot tub may increase the risk of eye infection from microorganisms.
- After the recommended wearing schedule, always discard lenses worn as prescribed by the Eye Care Professional.

Solution Precautions

- Different solutions cannot always be used together and not all solutions are safe for use with all lenses. Use only recommended solutions.
- The patient should not change solutions without consulting their Eye Care Professional.
- Never use solutions recommended for rigid gas permeable (RGP) contact lenses.
- Always use fresh, unexpired lens care solutions and lenses.
- Always follow directions in the package inserts for the use of contact lens solutions.
- Use only a chemical (not heat) lens care system. Use of a heat (thermal) care system can damage ACUVUE® and SUREVUE® Brand Contact Lenses.
- Sterile unpreserved solutions, when used, should be discarded after the time specified in the directions.
- Do not use saliva or anything other than the recommended solutions for lubricating or wetting lenses.
- Always keep the lenses completely immersed in the recommended storage solution when the lenses are not being worn (stored). Prolonged periods of drying will reduce the ability of the lens surface to return to a wettable state. Follow the lens care directions in "Care for a Dried Out (Dehydrated) Lens" if lens surface does become dried out.

Lens Case Precautions

Lens cases can be a source of bacterial growth and require proper use, cleaning and replacement at regular intervals as recommended by the lens case manufacturer or Eye Care Professional.

Other Topics to Discuss with Patients

- Always contact the Eye Care Professional before using any medicine in the eyes.
- Certain medications, such as antihistamines, decongestants, diuretics, muscle relaxants, tranquilizers and those for motion sickness may cause dryness of the eye, increased lens awareness or blurred vision. Should such conditions exist, proper remedial measures should be prescribed. Depending on the severity, this could include the use of lubricating drops that are indicated for use with soft contact lenses or the temporary discontinuance of contact lens wear while such medication is being used.
- Oral contraceptive users could develop visual changes or changes in lens tolerance when using contact lenses. Patients should be cautioned accordingly.
- As with any contact lens, follow-up visits are necessary to assure the continuing health of the patient's eyes. The patient should be instructed as to a recommended follow-up schedule.

Who Should Know That the Patient is Wearing Contact Lenses?

- Inform the doctor (Health Care Professional) about being a contact lens wearer.
- Always inform the employer of being a contact lens wearer. Some jobs may require use of eye protection equipment or may require that the patient not wear contact lenses.

ADVERSE REACTIONS

The patient should be informed that the following problems may occur when wearing ACUVUE® or SUREVUE® Brand Contact Lenses:

- The eye may burn, sting and/or itch.
- There may be less comfort than when the lens was first placed on the eye.
- There may be a feeling of something in the eye (foreign body, scratched area).
- There may be the potential for some temporary impairment due to peripheral infiltrates, peripheral corneal ulcers and corneal erosion. There may be the potential for other physiological observations, such as local or generalized oedema, corneal neovascularization, corneal staining, injection, tarsal abnormalities, iritis and conjunctivitis, some of which are clinically acceptable in low amounts.
- There may be excessive watering, unusual eye secretions, or redness of the eye.
- Poor visual acuity, blurred vision, rainbows or haloes around objects, photophobia, or dry eyes may also occur if the lenses are worn continuously or for too long a time.

The patient should be instructed to conduct a simple 3-part self-examination at least once a day. They should ask themselves:

- How do the lenses feel on my eyes?

- How do my eyes look?
- Have I noticed a change in my vision?

If the patient reports any problems, he or she should be instructed to **IMMEDIATELY REMOVE THE LENS**. If the discomfort or problem stops, the patient should then look closely at the lens. If the lens is in any way damaged, the patient **SHOULD NOT** put the lens back on the eye. The patient should discard the lens and apply a new fresh lens on the eye.

If the lens has dirt, an eyelash, or foreign body on it, or the problem stops and the lens appears undamaged, he or she should be instructed to dispose of the lens and apply a new fresh lens. If the problem continues, the patient **SHOULD NOT** put the lens back on the eye but **IMMEDIATELY CONSULT HIS OR HER EYE CARE PROFESSIONAL**.

The patient should also be instructed **NOT** to use a new lens as self-treatment for the problem. The patient should be advised that when any of the above symptoms occur, a serious condition such as infection, corneal ulcer, neovascularization or iritis may be present. The patient should be instructed to seek immediate professional identification of the problem and prompt treatment to avoid serious eye damage.

During therapeutic use an adverse effect may be due to the original disease or injury or may be due to the effects of wearing a contact lens. There is a possibility that the existing disease or condition might become worse when a soft contact lens for therapeutic use is used to treat an already diseased injured eye. The patient should be instructed to avoid serious eye damage by contacting the Eye Care Professional **IMMEDIATELY** if there is an increase in symptoms while wearing the lens.

Lens care directions

When lenses are dispensed, the Eye Care Professional should provide the patient with appropriate and adequate warnings and instructions in accordance with the individual patient's lens type and wearing schedule. The Eye Care Professional should recommend an appropriate care system tailored to the patient's individual requirements.

Failure to follow the correct lens care regime may result in serious injury to the eye, as described in the section entitled – **“Warnings”**

For complete information concerning contact lens handling, care, cleaning, disinfecting and storage, refer to the Patient Instruction Guide for ACUVUE® or SUREVUE® Brand Contact Lenses.

When ACUVUE® or SUREVUE® Brand Contact Lenses are recommended for frequent replacement, as shown in **Table 1**, the lenses must be cleaned and disinfected following removal before re-use. The lenses may be disinfected using a chemical disinfection system only (e.g. multi-purpose or hydrogen peroxide system).

The Eye Care Professional should review with the patient lens care directions, including basic information on lens case cleaning as well as specific instructions on the lens care regimen recommended for the patient. Since some lens materials contain silicone, as shown in **Table 1**, the wettability may differ when different lens care products are used.

Care for a sticking (non-moving) lens

If the lens sticks (stops moving), the patient should be instructed to apply a few drops of a recommended lubricating or rewetting solution directly to the eye and wait until the lens begins to move freely on the eye before removing it. If non-movement of the lens continues after a few minutes, the patient should immediately consult the Eye Care Professional.

Care for a dried out (dehydrated) lens

If any ACUVUE® or SUREVUE® Lens is off the eye for a prolonged period, its surface may become dry and gradually become non-wetting. If this should occur, discard the lens and use a new one.

EMERGENCIES

The patient should be informed that if chemicals of any kind (household products, gardening solutions, laboratory chemicals, etc.) are splashed into the eyes, the patient should: **FLUSH EYES IMMEDIATELY WITH TAP WATER AND IMMEDIATELY CONTACT THE EYE CARE PROFESSIONAL OR VISIT A HOSPITAL CASUALTY DEPARTMENT WITHOUT DELAY.**

REPORTING OF ADVERSE REACTIONS

All serious adverse experiences and adverse reactions observed in patients wearing ACUVUE® and SUREVUE® Brand Contact Lenses or experienced with the lenses should be reported to:

Johnson & Johnson Medical Limited
Pinewood Campus
Nine Mile Ride
Wokingham
RG40 3EW
United Kingdom
Tel: 0800 022 4222 Fax: 0800 783 7029
E-mail: ukcs@visgb.jnj.com

Additional information

For additional information concerning ACUVUE® or SUREVUE® Brand Contact Lenses and to order your free copy of ACUVUE® or SUREVUE® Brand Contact Lenses Patient Instruction Guides, please contact Customer Services on the address shown above.

Manufactured by:

Please refer to carton for site of manufacture

USA:

Johnson & Johnson Vision Care, Inc.
7500 Centurion Parkway
Jacksonville
Florida, 32256
USA

IRELAND:

Johnson & Johnson Vision Care (Ireland)
The National Technology Park
Limerick
Ireland

EU Authorised Representative:

Johnson & Johnson Medical Limited
Pinewood Campus
Nine Mile Ride
Wokingham
RG40 3EW
United Kingdom
www.acuvue.com

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©Johnson & Johnson Medical Limited. 2012
ACUVUE®2®, 1•DAY ACUVUE®, 1•DAY ACUVUE® MOIST®, SUREVUE®, ACUVUE® ADVANCE® with HYDRACLEAR®, ACUVUE® OASYS® with HYDRACLEAR® PLUS, 1•DAY ACUVUE® TruEye®, ACUVUE® ADVANCE® PLUS with HYDRACLEAR®, 1•DAY ACUVUE® DEFINE™ with LACREON™, are trademarks of Johnson & Johnson Medical Limited
AB Yetkili Temsilcisi: